


MODE D'EMPLOI FIX-IV

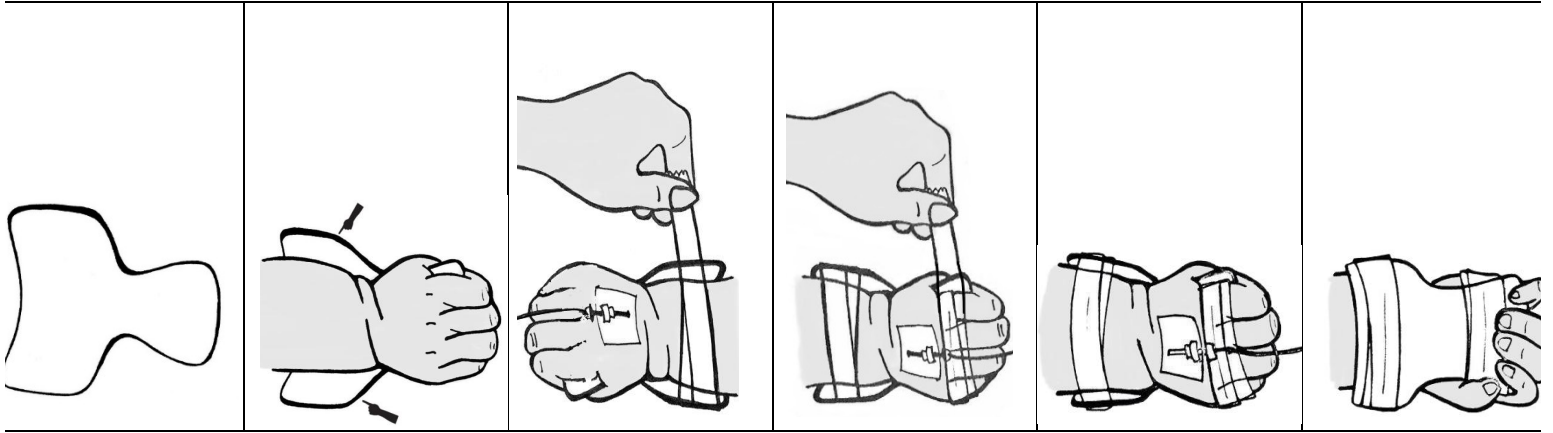

Mode d'emploi

CHOISIR la bonne dimension de l'attelle (Voir charte). La peau doit être en contact avec le coté rembourré et troué.

FIXER solidement la voie intraveineuse. **PLACER** l'attelle sous la main en orientant les ailettes chaque coté du poignet

COMMENCER par fixer le poignet. Ramener les ailettes chaque côté du poignet et fixer avec bande adhésive

APPLIQUER une autre bande adhésive au niveau des doigts. (Phalanges proximales).

S'ASSURER que les bandes adhésives soient assez longues pour un maintien efficace.

TERMINER en fixant tubulure l'attelle avec une bande adhésive

Instructions for use

ELECT the correct splint size (see chart). Make sure the skin is against the padded side with the holes.

SECURE the intravenous route. Place the splint under the hand, with the flaps on either side of the wrist.

START by immobilizing the wrist. Bring the flaps up against either side of the wrist and secure them with strips of adhesive tape.

PLACE another strip of adhesive tape across the fingers (proximal phalanges).

MAKE SURE that the adhesive strips are long enough to hold securely.

END by securing the tubing against the splint.